

Universal Medicap Ltd.



TestPac

TM

Universal Virus Transport Kit

- TestPac Universal Virus Transport Kit is intended for the collection and transport of clinical specimens from the collection site to the testing laboratory
- Recommended storage temperature of 2°C-30°C
- European CE certified
- Dacron Polyester Head
- Sterile filtered medium








SCOPE OF USE

It is used for the collection, transportation and preservation of oropharyngeal and nasopharyngeal parts of respiratory viruses and enteric viruses such as new coronavirus (COVID-19), influenza, bird flu, hand-foot-mouth, swine flu and so on. It is also suitable for the collection of other viruses, such as chlamydia, mycoplasma, and ureaplasma urealyticum specimens.



INSTRUCTION FOR USE

- Step 1**
 Tear off the package and take out the swab
- Step 2**
 Get sampling with swab from right body parts
- Step 3**
 Place the swab into virus specimen tube all the way to the bottom of the tube
- Step 4**
 Bend the swab at a 180 degrees angle to break at the appropriate point
- Step 5**
 Tighten the cap, shake it several times and complete the sampling

NOTE: After collection, the specimen should be transported to the laboratory within 72 hours with recommended storage temperature of 2-8°C. If delivery and processing exceed 72 hours, specimens should be transported in dry ice and once in laboratory frozen at -70 degree C or colder. The specimens should avoid repeated freezing and thawing.

PRODUCT CODE	PRODUCT DESCRIPTION	PACK SIZE	STORAGE	SHELF LIFE
UMLUVTK01	Disposable Virus specimen collection tube (3ml) with two swabs	Packing: 50 tubes/Box 100 tubes/Box	Stored at 2°C~30°C	1 year

*Please Note: Product Specification are subject to change without prior notice owing to product modification improvement /up-gradation.

*The product is manufactured & marketed by Universal Medicap Limited



Universal Medicap Ltd.

'UML House', Parag Park, At. Dashrath,
Dist.: Vadodara - 391 740, Gujarat, India.

Tel.: +91-265-2243055, 2240004, 2240230, 99789 51002

www.universalmedicap.com | E-mail: marketing@umlindia.com